

Power Clean Vacucid®



Vacucid® Power Clean is a concentrated cleaning solution containing 5 enzymes (amylase, cellulase, lipase, mannanase and protease). Designed for fast and effective cleaning of dental suction systems. Its powerful action is efficient on the breakdown of organic debris, preventing the accumulation of bioburden.

Key product features

- Economical 2% dilution
- Highly effective on organic soil
- Contact time: 15 minutes
- Designed for daily use with all makes of suction system
- Proven efficiency on biofilm in just 5 minutes
- Non-foaming
- Maintains cleaning properties in all qualities of water (softened, mains etc)
- VOC free formulation with a neutral pH when diluted
- CE Marked – Complies with Medical Device Directive 93/42/EEC



USF Healthcare S.A. | Rue François Perréard 4 | CH-1225 Chêne Bourg | Geneva | Switzerland
Tél + 41 22 839 79 00 | Fax + 41 22 839 79 10 | info@usfhc.com
www.usfhealthcare.com



Power Clean Vacucid®


Protocol for use

Before use

Ensure that appropriate gloves are worn.




Using the Vacucid® Systematic, dilute at 2% (40ml for 2 litres of solution).

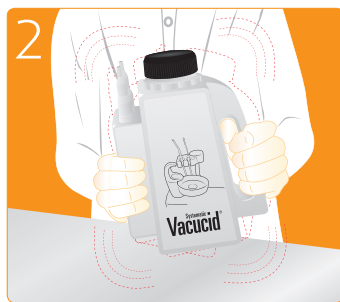

Gently shake the Vacucid® Systematic to mix the solution.



Attach the Vacucid® Systematic to the suction system and aspirate the diluted solution.

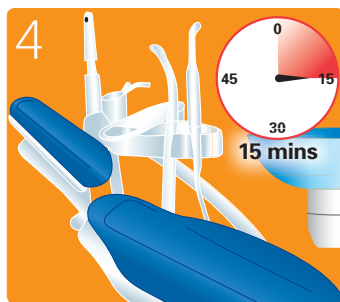

Leave the solution to work for a minimum of 15 minutes. Solution can be left in system overnight.



Fill the Vacucid® Systematic with clean water.



Flush by sucking the clean water through the system (minimum 1 litre).

Product composition

Non-ionic and anionic surfactants, sequestering agent, enzyme complex, excipients.

Physical data

Colour: Blue
Perfume: Non-perfumed
pH: Approx. 8 (pure product)
Approx. 7.25 (diluted)
Shelf life: Concentrate 2 Years
Container: Made of recyclable HDPE material

Precautions

Dangerous: respect the precautions for use (established according to the European regulations in force for classification and labeling of chemicals). Before each use, read the label and the information concerning the product. Storage between +5°C and +35°C. Reserved for professional use. Medical device Class I (complies with medical devices directive 93/42/EEC modified).

Packaging

1kg	1L	2,5L	5L
			✓

Accessories

Vacucid® Systematic (424051)
5 litre container with dosing pump



ECOEngage
Our Ecological Engagement



USF Healthcare
Unident • Steridine • Farmec